

# UPGRADE SYSTEM EEG-1200 TO EEG-1260

## ISO and PC Installation



Before starting the upgrade, **verify that the Serial Number (SN) sticker on the new EEG-1260 PC matches the ISO.** If the serial numbers do not match, stop immediately and contact Technical Support at **1-800-325-0283**, following the prompts.



**IMPORTANT:** All systems upgraded from EEG-1200 to EEG-1260 must have the ISO replaced.


## ISO Replacement Steps



**01.** Power off the system and disconnect all devices from the back of the ISO.



**02.** Loosen the screw on the bottom ISO bracket, slide the bracket out, and remove the old ISO.

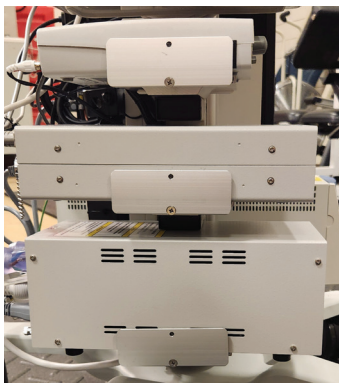


**03.** Position the new ISO on the bracket, slide the bracket plate forward, and tighten the screw to secure the ISO.



**04.** Reconnect all devices that were unplugged from the old ISO.


## PC Replacement Steps

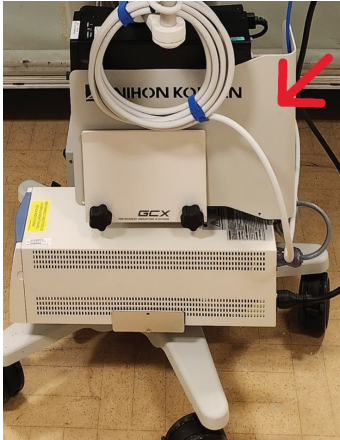

- 01.** Remove the side plate by loosening the two black knobs. Set the plate and knobs aside for later reinstallation. This provides easier access to the PC ports.



- 02.** Place the new EEG-1260 PC on the PC bracket and reconnect all devices to the appropriate ports.

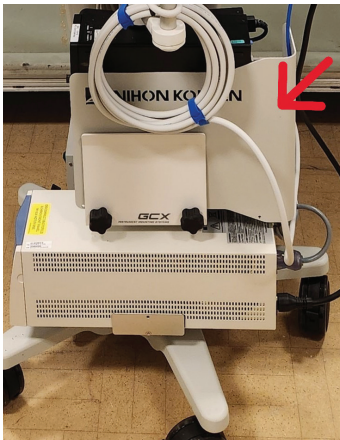

- 03.** Reinstall the side plate and secure it by tightening the black knobs.

**For more information, please contact us at 800-325-0283 or visit [us.nihonkohden.com](http://us.nihonkohden.com).**

### IMPORTANT SAFETY INFORMATION:

CAUTION: Federal (United States) law restricts this device to sale by or on the order of a physician. See Instructions for Use for full prescribing information, including indications, contraindications, warnings, precautions, and adverse events.



**NIHON KOHDEN AMERICA, LLC**

15353 Barranca Parkway, Irvine, CA 92618 U.S.A.

**1.800.325.0283 [us.nihonkohden.com](http://us.nihonkohden.com)**