

NK-HEALTHPROTECT™

WIRELESS SURGE SOLUTION

Setup Manual

Version 1 | © 2020 All Rights Reserved

Safeguarding Your Healthcare Professionals™



TABLE OF CONTENTS

NK-HealthProtect™ Wireless Surge Solution Setup Manual	1
Purpose	1
Scope	1
Definitions	1
Equipment Needed for First Setup	1
Procedure	2-3

TABLE OF FIGURES

Figure 1 NK-HealthProtect Wireless Surge Solution Wireless Surge Solution Kit Contents.....	2
Figure 2 Connect Ethernet Cable to Access Point	3
Figure 3 Install the AA Batteries into the Life Scope® G3 Series GZ-140 Transmitter	3

NK-HEALTHPROTECT™

WIRELESS SURGE SOLUTION SETUP MANUAL

Purpose

The purpose of this setup manual is to show how to setup the NK-HealthProtect Wireless Surge Solution and connect the equipment together.

Scope

This manual applies to both the patient monitoring devices and the Prefense® Laptop. It applies to the setup of the devices only.

Equipment Needed

PART NUMBERS	DESCRIPTION	QUANTITY
GZ-140PA	Life Scope® G3 Series GZ-140 Telemetry Transmitter	8
A/Prefense-WSS	Prefense Software	1
A/Hotspot	USB Access Point with included Ethernet Cable	1
A/V6KH977	Laptop	1
A/4881351	AA Lithium Batteries – 8 Pack	6

Table 1

Procedure

NK-HealthProtect™ Wireless Surge Solution



Figure 1 Wireless Surge Solution Kit Contents

1. Begin by removing the devices and accessories from the hard-shell case.
2. Refer to Table 1 (Page 1) for a list of the parts required for setup.
3. Connect the laptop power supply to the laptop and an AC power source.
4. Plug the access point's ethernet cable into the ethernet port on the laptop.
5. Plug the access point's USB cable into a USB port on the laptop. The access point will receive power from the laptop.



Figure 2
Connect Ethernet Cable to Access Point



Figure 3
Install the AA Batteries into the
Life Scope G3 Series GZ-140 Transmitters

6. Power on the laptop. Wait two (2) minutes before going on to the next step. The Prefense® software application will begin running automatically.
7. Install the AA batteries into the Life Scope® G3 Series GZ-140 devices as shown in Figure 3.
8. (Optional) If you have ordered a printer, remove the printer from the box, connect the power cord to the printer and plug the USB cable into the laptop. The drivers are preinstalled on the laptop. The printer will be ready to use.
9. Power on the Life Scope G3 Series GZ-140 devices.
10. The Life Scope G3 GZ-140 devices will automatically connect to the system. Data from the devices will then be viewable in Prefense.
11. For additional assistance and support call 800.325.0283 or email VitalSignsTechSupport@nihonkohden.com. Additional support information is also available at <https://nkuniversity.org>.

Safeguarding Your Healthcare Professionals™



**For more information, please contact us at
1-800-325-0283 or visit us.nihonkohden.com**

Nihon Kohden University is a registered trademark of Nihon Kohden.

Life Scope is a registered trademark of Nihon Kohden Corporation.

Prefense is a registered trademark of Nihon Kohden America.

NK-HealthProtect, Safeguarding Your Healthcare Professionals, and Exceptional Solutions, Simply Delivered are trademarks of Nihon Kohden.



**Exceptional Solutions,
Simply Delivered.™**

MTSI 049 (B)-CO-3221